

# SAREMCO Dental restorative system with cmf

1



cmf etch



etch  
total-etch



rinse  
thoroughly




dry  
with oilfree  
air, keep dry

etch


2



cmf prime

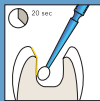

prime  
apply and  
massage

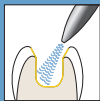


dry  
with oilfree,  
air, keep dry


prime and  
bond



cmf bond

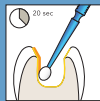

bond  
apply and  
massage

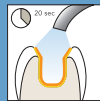

cure

# 3




els flow




els extra low shrinkage®



line and cure  
(els flow optional)

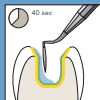

restore and cure  
layer by layer

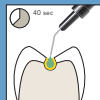

restore and cure  
(small cavities)  
els flow

## restore

Exposure time for increments of 2mm

light power	≥ 500 mW/cm <sup>2</sup>	≥ 800 mW/cm <sup>2</sup>
els incisal colours	20 sec.	20 sec.
els opaque colours	40 sec.	30 sec.
els all other colours	40 sec.	20 sec.