

SAREMCO Dental ceramic silane primer

1



silane base and
silane activator



roughen
surface,
keep dry



dose
2 drops base,
1 drop
activator




mix
well




wait
until liquid appears
homogenous
and transparent

prepare

2

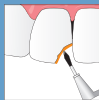

apply

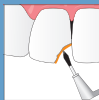

massage
to a lucent
film



dry
with oilfree
air, keep dry

pass immediately to step 3

prime

3



cmf bond



bond
apply and
massage



cure

bond

4



els extra low
shrinkage®



restore and
cure
layer by layer



connect
inlay and
cure

restore