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1. Produktbeschreibung

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Instruction for use james-2

1. Product description

Light-curing, solvent-free 1-component restorative adhesive in the total-etch process for enamel and dentine.

2. Composition

hydroxyethylmethacrylate, urethanemethacrylate, polyalkenoate methacrylized, hydroxypropylmethacrylate, glycerinedimethacrylate, catalysts, inhibitors

3. Indication

- Direct restorations of all cavity classes on anterior and posterior teeth with light-curing composites
- Direct restorations and core build-ups with dual-curing or self-curing composites
- Adhesive bonding of indirect restorations (crowns, bridges, inlays, onlays, veneers) with light-curing, dual-curing or self-curing composite cements

4. Contra-indication

Opened pulp, pulpitis, known allergy to methacrylates.

5. Side effects

In individual cases, contact allergies have been described to products of a similar composition. To avoid pulp reactions, it is recommended to cover the exposed dentine in the pulp area with suitable underfilling materials (preferably with calcium hydroxide).

6. Interactions

Avoid underfilling materials which may hinder polymerization owing to their ingredients. All phenolic compounds, such as zinc oxide eugenol or preparations containing thymol, belong to this category.

7. Processing stages

7.1. Drying

Due to the following application of the filling material, a drainage of the working field is required. Avoid any contamination of the cavity with blood or saliva. A rubber dam is recommended.

7.2. Cavity preparation

Prepare the cavity as usual. Undercuts and bevelled margins are recommended to improve adhesion and the margin fit. Clean and dry the cavity. Avoid overdrilling.

7.3. Etching of enamel and dentine

Apply the etching gel and condition in correspondence to instruction for use of the product. SAREMCO recommends microcid and cmf etch. Sufficiently etched enamel appears matt and white.

Note: Possibly use the total-etch-technique in cavities limited by dentine. In this case, the dentine should not be etched for more than 15 seconds and only dried briefly after rinsing.

7.4. Application

Apply james-2 to enamel/dentine in a not too thin layer with a brush and massage into the tooth substance for 20 seconds. Note: james-2 is light-curing, so avoid too intense ambient light. Dim the surgical light during the application.

7.5. Drying

Dry the adhesive layer with an air blower for at least 5 seconds.

7.6. Polymerizing

Polymerize the bonding layer with blue light (halogen or LED lamp) for 30 seconds. The specified exposure time relates to halogen or LED light-curing devices with a minimum light intensity of 500 mW/cm² and a wavelength of 400 - 500 nm.

Note: Do not use james-2 for opaque restorations or deep cavities where complete light-curing is not guaranteed.

Application and processing of the composite filling material according to the instructions for use of the product used. SAREMCO recommends mh composite, els composite and apt composite.

8. Storage

Close james-2 bottle tightly after use. Store in an upright position to ensure the backflow of the liquid. Do not expose light-curing products to direct sunlight or operating light. james-2 was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Constant temperatures above 28°C / 82°F can reduce the shelf-life of the product.

9. Batch number and expiry date

The batch number should be specified to identify products in the case of enquiries. Products should no longer be used once the expiry date has elapsed.

10. Precautionary measures

Close containers after each use with the right lid. Keep out of reach of children. Commercially available medical gloves do not provide protection against the sensitisation effect of methacrylates. If the product comes into contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice.

11. Emergency measures

In case of direct contact with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an eye specialist.

12. Hygiene

Use application instruments for one patient only. Dose products away from patients to avoid contamination.

13. Warranty

Our liability is restricted to the quality of our products. In case of a product being of defective quality, only its value is replaced. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental.

14. Production / distribution

SAREMCO Dental AG
Gewerbestrasse 4
CH-9445 Rebstein / Switzerland
Tel: +41 (0) 71 775 80 90
Fax: +41 (0) 71 775 80 99
info@saremco.ch
www.saremco.ch

Edited 10/2020 | D600170

Class IIa medical devices
Class III medical devices (Canada)

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