Gebrauchsanweisung els paintart

1. Produktbeschreibung

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Instruction for use els paintart

1. Product description

Light-curing, low viscous color resin for the characterization of restorations in esthetic dentistry. Available in the colors incisal, white, yellow, blue, red, brown and grey.

2. Composition

BisGMA, BisEMA, highly dispersed silicium dioxide, silanized, catalysts, inhibitors, color pigments

3. Intended Use

Saremc restoratives are intended for the direct reconstruction or correction of functionally compromised natural dentition (e.g., deficient teeth).

4. Indication

Color characterization of composite restorations.

5. Contra-indication

Known allergy to methacrylates, camphorquinone and BHT.

6. Side effects

In individual cases, contact allergies have been described to products of a similar composition. **Note: els paintart** does not contain TEGDMA or HEMA.

7. Interactions

None known.

8. Processing stages

8.1. Application

Attach the application needle. Apply the product directly from the syringe or with a brush onto the cured or uncured composite filling (SAREMCO recommends **apt composite** and **els composite**) and light cure.

Warning: Remove the application cannula after each use and replace the cap. Protect light-curing products from strong sources of light.

The specified exposure times refer to halogen or LED light-curing devices with a minimum light intensity of 500 mW/cm² and a wavelength of 400 - 500 nm. They apply to a maximum layer thickness of 0,5 mm and for all colors. The necessary exposure time may vary depending on the light source and its instructions for use. If in doubt, check the lamp's light output and the necessary exposure time before operation in vitro.

Exposure time for increments of 0.5 mm

light power	≥ 500 mW/cm ²	≥ 1'200 mW/cm ²
els paintart all colors	40 sec	20 sec

8.2. Polishing

High gloss polishing with polishing brushes, polishing discs, strips or silicone polishers.

9. Storage

Protect light-curing products from strong sources of light and heat! **els paintart** was developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Bring to room temperature before use.

10. Batch number and expiry date

The batch number should be specified to identify products in case of enquiries. Products should no longer be used once the expiry date has elapsed.

11. Precautionary measures

Close containers after each use with the right lid. Keep out of reach of children. For dental use only. Commercially available medical gloves do not provide protection against the sensitization effect of methacrylates. If the product comes in contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice. When polishing or removing composites, it is recommended to always use a water-cooling system and a good extraction system, to ventilate the dental laboratory frequently and to wear masks with high particle filtration efficiency for small particle sizes.

12. Emergency measures

The product was developed with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an ophthalmologist.

13. Hygiene

Use application instruments for one patient only. Dose products away from patients to avoid contamination. Sterilization of syringes or tips before first use is not required.

14. Warranty

The product was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. Our liability is restricted to the quality of our products. In the case of a product being of defective quality, only its value is replaced. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental AG.

15. Other notes for Europe

If the user and/or patient become aware of serious incidents connected with application the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides.

16. Production / distribution

SAREMCO Dental AG
Gewerbstrasse 4
CH-9445 Rebstein / Switzerland
Tel: +41 (0) 71 775 80 90
Fax: +41 (0) 71 775 80 99
info@saremc.ch
www.saremc.ch

Edited 02/2022 | D600174

Class IIa medical devices
Class III medical devices (Canada)

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