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1. Produktbeschreibung

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Instruction for use apt unibond

1. Product description

Light-curing, 1-component self-etching adhesive to create a permanent marginal-gap-free adhesion between the tooth structure (dentine, enamel) and the light-curing filling/fixing material.

2. Composition

ethanol, methacrylated phosphoric salt, aromatic urethanmethacrylate, BisEMA, water, fillers, initiators

3. Intended Use

SAREMCO dental bonding agents promote adhesion of direct or indirect prosthesis for reconstruction or correction of functionally compromised natural dentition (e.g., deficient teeth).

4. Indication

- Direct restorations with light-curing composite-based materials.
- Indirect restorations: at the use of light-curing composite cements to fix inlays, onlays, crowns and bridges.

5. Contra-indication

Opened pulp, pulpitis, known allergy to methacrylates.

6. Side effects

In individual cases, contact allergies have been described to products of a similar composition. To avoid pulp reactions, it is recommended to cover the exposed dentine in the pulp area with suitable underfilling materials (preferably with calcium hydroxide). **Note:** apt unibond does not contain TEGDMA or HEMA.

7. Interactions

Avoid underfilling materials which may hinder polymerization owing to their ingredients. All phenolic compounds, such as zinc oxide eugenol or preparations containing thymol, belong to this category.

8. Processing stages

8.1. Drying

Due to the following application of the filling material, a drainage of the working field is required. Avoid any contamination of the cavity with blood or saliva. A rubber dam is recommended.

8.2. Cavity preparation

Prepare the cavity as usual. Undercuts and bevelled margins are recommended to improve adhesion and the margin fit. Clean and dry the cavity. Avoid overdrilling.

8.2.1. Etching of tooth substance (optional)
By applying apt unibond clinically sufficient bond strength values are achieved. By selective enamel etching or in total-etch technique even higher adhesion values can be achieved.

8.2.1.1 Total-etch technique (with SAREMCO cmf etch)

Apply the etching gel and condition in correspondence to instruction for use of the product. SAREMCO recommends cmf etch.

8.2.1.2 Selective enamel etching

Apply the etching gel and condition in correspondence to instruction for use of the product. SAREMCO recommends cmf etch. Sufficiently etched enamel appears matt and white. **Note:** Possibly use the total-etch-technique in cavities limited by dentine. In this case, the dentine should not be etched for more than 15 seconds and only dried briefly after rinsing.

8.2.2 Non-Etch

Do not overdry the dentine. A slightly moist cavity is optimal. Without preparation (e.g., cervical defect), thoroughly clean the surface and roughen the enamel margin with a finishing diamond.

8.3. Application

Apply apt unibond to enamel/dentine in a not too thin layer with a brush and massage into the tooth substance for 20 seconds. **Note:** apt unibond is light-curing, so avoid too intense ambient light. Dim the surgical light during the application.

8.4. Drying

Dry the adhesive layer with an air blower for at least 5 seconds.

8.5. Polymerizing

Polymerize the bonding layer with blue light (halogen or LED lamp). The specified exposure time refers to halogen or LED light-curing devices with a minimum light intensity of 500 mW/cm² and a wavelength of 400 - 500 nm. The necessary exposure time may vary depending on the light source and its instructions for use. In case of doubt, check the lamp's light output and the necessary exposure time before operation in vitro. Bring the light curing device as close as possible to the surface to be polymerized. During polymerisation, an inhibition layer forms on the surface; this must not be touched or removed, since it is important for the bond with the subsequent composite layers.

Exposure time

light power	≥ 500 mW/cm ²	≥ 1'200 mW/cm ²
exposure time	20 sec	10 sec

Important: Extending the exposure times does not have a positive effect on the adhesion! When using dual-curing luting composites, light curing of the luting composite is absolutely necessary.

Note: Do not use apt unibond for opaque restorations or deep cavities where complete light-curing is not guaranteed.

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