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CE 0123

Instruction for use cmf adhesive system

1. Product description

Light-curing, 2-step adhesive to create a permanent marginal-gap-free adhesion between tooth structure (dentin/enamel) and dental filling composite with total etch-technique.

cmf adhesive system consists of **cmf prime** for conditioning of dentin and **cmf bond** for bonding between conditioned dentin, etched enamel and composite. **cmf adhesive system** tolerates residual moisture and “wet bonding”.

2. Composition

cmf prime: water, alcohol, acetone, met-hacrylated phosphoric salt, catalysts, inhibitors
cmf bond: BisEMA, barium glass, silica, cata-lysts, inhibitors

3. Intended use

Saremco dental bonding agents promote adhesion of direct or indirect prostheses for reconstruction or correction of functionally compromised natural dentition (e.g. deficient teeth).

4. Indication

1. Restoration of all cavity-classes on anterior and posterior teeth with light-curing composite (e.g. els composite, els flow)
2. Direct restorations of core build-ups with dual- and self-curing composites
3. Adhesive attachment of indirect restorations (crowns, bridges, inlays, onlays, veneers) with light-, dual-, or self-curing composite cements (els cem)

5. Contra-indication

Opened pulp, pulpitis, known allergy to methacrylates.

6. Side effects

In individual cases, contact allergies have been described to products of a similar composition. To prevent reaction, it is advisable to cover the exposed dentine in the pulp area with suitable underfilling materials (preferably with calcium hydroxide). **Note:** **cmf adhesive system** does not contain TEGDMA, HEMA or BisGMA.

7. Interactions

Avoid underfilling materials which may hinder polymerisation owing to their ingredients. All phenolic compounds, such as zinc oxide eugenol or preparations containing thymol, fall under this category. Temporary attachments should be carried out with an eugenol-free temporary cement.

8. Processing stages

8.1. Preparation

8.1.1. Tooth cleaning
Brush the tooth being treated and the teeth next to it with fluoride-free toothpaste. If necessary, clean interdental spaces using strips and dental floss.

8.1.2. Shape selection

SAREMCO shades relate to VITA standards.

8.1.3. Drying

Dry sufficiently, apply rubber dam.

8.1.4. Cavity preparation

Prepare the cavity as usual. Undercuts and bevelled margins are recommended to improve adhesion and margin fit. Clean and dry the cavity. Avoid overdryng.

8.1.5. Underfilling

In case of near-to-pulp preparations, it is advised to protect the pulp using an underfilling (e.g. calcium hydroxide). Do not use zinc oxide eugenol!

8.2. Total-etch-technique (with cmf etch)

Apply cmf etch to enamel and dentine. Allow to have an effect on enamel and dentine for 30 seconds. Rinses and dry thoroughly. Avoid overdryng.

8.3. Application cmf adhesive system

8.3.1. cmf prime

Apply **cmf prime** to the cavity and work it in for 20 seconds. Light cure for 20 seconds. One layer is sufficient. Do not touch or contaminate the inhibition layer after polymerisation to ensure a reliable adhesion with the composite.

8.3.2. „Bond“ with cmf bond

Apply **cmf bond** to the cavity and work it in for 20 seconds. Light cure for 20 seconds. One layer is sufficient. Do not touch or contaminate the inhibition layer after polymerisation to ensure a reliable adhesion with the composite.

8.4. Restoration

8.4.1. Direct Restoration with composites (els composite / els flow)

Apply composite filling material according to the manufacturer's instructions and adapt to the cavity walls. Build up the composite filling layer by layer with a maximum layer thickness of 2 mm, polymerise according to the manufacturer's instructions and finalise the restauration.

8.4.2. Adhesive fixation of indirect restorations with light-, dual- or self-curing composite-cements (els cem)

Preparation: see 7.1. Remove cement residues, temporary luting materials and protective varnishes mechanically from the relevant teeth. Prepare inner surfaces of restorations according to the manufacturer's instructions (sandblast, silanize, etc.) Afterwards avoid any contamination of the surfaces.

Etching process:

analogue 7.2.

Application:

analoge 7.3.

In well-fitting restorations make sure, that in any area of the tooth surface the adhesive will not be polymerised in too thick layers.

Cementing:

application: is possible with chemically or light-curing composite luting cements. Process cement according to manufacturer's instructions and apply to the restoration. Finally, insert and fix the restoration.

Storage

Close **cmf** bottles tightly after use. Store in an upright position to ensure the backflow of the liquid. Do not expose light-curing products to direct sunlight or operating light. **cmf prime** and **cmf bond** were developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Constant temperatures above 28°C / 82°F can reduce the shelf-life of the products.

Batch number and expiry date

The batch number should be specified to identify products in the case of enquiries. Products should no longer be used once the expiry date has elapsed.

11. Precautionary measures

Close containers after each use with the right lid. Keep out of reach of children. For dental use only. Commercially available medical gloves do not provide protection against the sensitisation effect of methacrylates. If the product comes into contact with the glove, remove the glove and dispose of it, wash your hands with water and soap immediately and put on a new glove. In case of an allergic reaction, seek medical advice. When polishing or removing composites, it is recommended to always use a water-cooling system and a good extraction system, to ventilate the dental laboratory frequently and to wear masks with high particle filtration efficiency for small particle sizes.

12. Emergency measures

In case of direct contact with the oral mucosa, rinse with water. In case of contact with the eyes, rinse thoroughly with water. Consult an eye specialist.

13. Hygiene

Use application instruments for one patient only. Dose products away from patients to avoid contamination.

14. Liability

The product was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other inappropriate handling or inappropriate use of a product, any liability is restricted to the quality of our products. In the case of a product being of defective quality, only its value is replaced. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental AG.

15. Other notes for Europe

If the user and/or patient becomes aware of serious incidents in connection with the use of the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides.

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cmf prime: water, alcohol, acetone, metacrylated phosphoric salt, catalysts, inhibitors
cmf bond: BisEMA, barium glass, silica, catalysts, inhibitors

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