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## Instruction for use microcid etchant gel

### 1. Product description

Etching gel containing phosphoric acid for the selective enamel or total etching technique.

### 2. Composition

water, phosphoric acid (35 % wt), gel former, colorant

### 3. Intended use

Saremco dental bonding agents promote adhesion of direct or indirect prostheses for reconstruction or correction of functionally compromised natural dentition (e.g., deficient teeth).

### 4. Indication

Acid etching of enamel and dentin surfaces prior to the application of primers, adhesives, or sealers, etc.

### 5. Contra-indication

Opened pulp, pulpitis, known allergy to one or more ingredients.

### 6. Patient target group

microcid etchant gel is suitable for use in all patients without any age or gender restrictions.

### 7. User

microcid etchant gel should only be used by a professionally trained dental practitioner.

### 8. Side effects

In individual cases, contact allergies have been described to products of a similar composition. To avoid pulp reactions, it is recommended to cover the exposed dentine in the pulp area with suitable underfilling materials (preferably with calcium hydroxide).

### 9. Interactions

Underfilling materials based on glass ionomer and calcium hydroxide are etched by microcid etchant gel.

### 10. Processing stages

#### 10.1. Drying

Due to the following application of the filling material, a drainage of the working field is required. Avoid any contamination of the cavity with blood or saliva. A rubber dam is recommended.

#### 10.2. Cavity preparation

Prepare the cavity as usual. Clean and dry the cavity.

#### 10.3. Etching of tooth substance

**10.3.1.** Use a new syringe tip for each patient. Before using in the mouth, put a small amount of the gel on a cotton pad to familiarize yourself with the viscosity.

**10.3.2.** Apply microcid etchant gel to enamel. Leave to act for 30 - 60 seconds until the rising of air bubbles can be observed. Rinse and dry thoroughly. Avoid overdrying. Sufficiently etched enamel appears matt and white.

Note: Possibly use the total-etch-technique in cavities limited by dentine. In this case, the dentine should not be etched for more than 15 seconds and only dried briefly after rinsing.

#### 10.4. Adhesion

The adhesive has to be conditioned and applied in correspondence to the instruction for use of the product used. SAREMCO recommends james-2, els unbond, apt unbond and cmf adhesive system.

#### 10.5. Refilling the microcid etchant gel syringe 2.5 ml

If necessary, microcid etchant gel 2.5 ml can be refilled with microcid etchant gel 25 ml. To do that, remove the caps from both syringes. Place the adapter supplied firmly on the 25 ml syringe and on the 2.5 ml syringe and screw tight. Once the syringes are firmly in place, slowly withdraw the plunger of the 2.5 ml syringe to collect the desired amount of material. Avoid unintentional leakage of the material. Carefully separate the syringes from the adapter. When not in use, always keep the syringes closed with the caps.

#### 11. Storage

microcid etchant gel has been developed for use at room temperature (20°C - 25°C / 68°F - 77°F). Store at temperatures between 4°C - 28°C / 39°F - 82°F. If room temperature exceeds 28°C / 82°F storing in the refrigerator is recommended. Do not freeze! Bring to room temperature before use.

#### 12. Batch number and expiry date

The batch number should be specified to identify products in case of enquiries. Products should no longer be used once the expiry date has elapsed.

#### 13. Precautionary measures

Close containers after each use with the right lid. microcid etchant gel is corrosive! Avoid contact with skin, oral mucosa, eyes, clothes, metal etc. Keep out of reach of children. For dental use only. The principles of good personal and safe working practice apply.

#### 14. Emergency measures

In case of direct contact with skin, oral mucosa or eyes, rinse immediately and thoroughly under running water. In case of contact with the eyes, consult an eye specialist.

### 15. Hygiene

Use application instruments for one patient only. Dose products away from patients to avoid contamination.

### 16. Warranty

The product was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. Our liability is restricted to the quality of our products. In the case of a product being of defective quality, only its value is replaced. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages. Safety data sheets and technical data sheets are available on the website of SAREMCO Dental AG.

### 17. Other notes for Europe

If the user and/or patient becomes aware of serious incidents in connection with the use of the product, they are to be reported to the manufacturer and the responsible authorities of the state in which the user and/or patient resides.

The Summary of Safety and Clinical Performance of SAREMCO Products can be found in the European database on medical devices (EUDAMED—<https://ec.europa.eu/tools/eudamed>).

### 18. Production / distribution

SAREMCO Dental AG  
Gewerbstrasse 4  
CH-9445 Rebstein / Switzerland  
Tel: +41 (0) 71 775 80 90  
Fax: +41 (0) 71 775 80 99  
[info@saremco.ch](mailto:info@saremco.ch)  
[www.saremco.ch](http://www.saremco.ch)

Edited: 10-2022 | D600173

Class IIa medical devices  
Class III medical devices (Canada)

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