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Instruction for use saremco applicator

1. Product description
saremco applicator is suitable for direct application of restorative material to tips. The parallel lever movement allows simple, precise aplication, even with highly filled composites.

2. Intended use
Direct application of composite filling material from tips into the tooth cavity to be restored. Dosage of composite filling material from tips onto a mixing block for application using an instrument.

3. Indication
Used in combination with a range of medical devices.

4. Patient target group.
The saremco applicator can be used on patients regardless of any age or gender.

5. User
The saremco applicator should only be used by a professionally trained dental practitioner.

6. Use
a. Pull the quick-release fastener as far back as possible. b. Insert the tip into the saremco applicator and slowly release the fastener until the tip is securely locked into place.

c. To turn the tip, if necessary, the fastener should first be slightly pulled back again. Then remove the protective cap. Extrude the material directly into the cavity by slowly and evenly pressing the handle and lever together. Do not apply too much pressure.

d. After the material has been applied, retract the quick-release fastener and pull the tip forward and up to remove it.
e. Clean and disinfect the saremco applicator after use. The saremco applicator can be steam-sterilized up to 134°C (steam autoclave, follow the manufacturer’s instruction for use). In addition, ethanol can be used for cleaning.

7. Storage
Store at 4°C - 28°C / 39°F - 82°F.

8. Batch number and expiry date
The batch number is used to identify the product in case of queries. Do not use this product after the expiration date.

9. Precautionary measures
The composite or flow tips allow quick and hygienic direct application. The material can be precisely dispensed into the cavity through the small opening. Detailed information for each saremco product can be found in the individual instructions for use. Do not use visibly damaged devices.

Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

10. Hygiene
Only use saremco applicator for one patient at a time after cleaning and disinfection. Sterilization of syringes or tips before first use is not required.

11. Disposal
This product can be disposed of with household waste. Specific regulations or laws may apply depending on the country.

12. Warranty
The product was developed for use in dentistry and must be processed in accordance with the instructions for use. For further damages, namely that caused by non-compliance with the instructions for use or other improper handling or inappropriate use of a product, any liability is rejected. Our liability is restricted to the quality of our products. In the case of a product being of defective quality, only its value is replaced. It is the responsibility of the user to check, before using the products, whether they are suitable for the intended purpose. He expressly assumes all risks associated with using the product and is solely responsible for any resulting damages.

13. Other notes for Europe
If the user and/or patient becomes aware of serious incidents as a result of application of product, such serious incidents must be reported to the manufacturer and the responsible authorities of the country in which the user and/or patient resides.
The Summary of Safety and Clinical Performance of saremco applicator can be found in the European database on medical devices (EUDAMED – https://ec.europa.eu/tools/eudamed).

14. Production / Distribution
SAREMCO Dental AG
Gewerbestrasse 4
CH-9445 Rebstein / Switzerland
Tel: +41 (0) 71 775 80 90
Fax: +41 (0) 71 775 80 99
info@saremco.ch
www.saremco.ch

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